

365 Transparent Island Dressings



Product Description

365 Transparent Island Dressings are sterile, self-adhesive, transparent film wound dressings incorporating an absorbent, non-adherent pad. 365 Transparent Island Dressings feature a latex free, semi-permeable polyurethane film membrane which is coated with an hypoallergenic acrylic adhesive.

365 Transparent Island dressings are breathable, allowing good oxygen and moisture vapour exchange. It is showerproof and impermeable to liquids, bacteria, and viruses.

These properties enable:

- The facilitation of wound healing
- Provision of a 'moist wound healing environment'
- The patient to shower with the dressing in place
- Protection of newly forming tissue
- Prevention of contamination by dirt or bacteria, thus a reduced risk of wound infection

Indications For Use:

365 Transparent Island Dressings are primarily indicated to be used on acute wounds with low levels of exudate. 365 Transparent Island Dressings are therefore suitable for the management of:

- Superficial Cuts and abrasions
- Clean, closed, post-operative wounds
- Minor burns
- Donor sites
- Chronic wounds with low levels of exudate
- A fixation device for catheters

365 Transparent Island dressings are also part of the 365 Healthcare range of Film dressings and I.V.

Preparation:

Use existing clinical protocols to clean/debride the wound in preparation for the application of 365 Transparent Island Dressings.

Ensure that the skin surrounding the wound is clean, dry, and free from grease, soaps or detergents. Clipping of hair at the site may improve adhesion, shaving is not recommended.

Select a dressing that is of a suitable size and shape. Choose a dressing size large enough to provide a margin that adheres to dry healthy skin around the catheter or wound site.

Dressing Application:

- Peel the two white paper liners from the dressing to expose the absorbent pad and adhesive border.
- Position the absorbent dressing pad centrally over the wound.
- Gently press down the adhesive border onto the skin.
- As indicated by the blue arrows remover the transparent release film on the back of the dressing from the centre outwards.
- Smooth the entire dressing from the centre toward the edges using gentle pressure to enhance adhesion.



Dressing Wear Time:

The frequency of dressing change will depend on the clinical circumstances. 365 Transparent Island Dressings may be left in place for up to seven days. If signs of clinical infection are observed the dressing should be removed.

Dressing Removal:

- To remove 365 Transparent Island Dressings from the wound, gently peel and lift one corner of the dressing from the skin.
- Support the skin whilst peeling the dressing off by stretching the film horizontally (not vertically) and in the direction of hair growth.
- Care should be taken to avoid skin damage with repeated applications, or on patients with fragile skin.
- Use existing clinical protocols to clean the wound in order to remove any remaining exudate before wound assessment or the application of further dressings

Ordering Information

	365 Film	Product Description	Box quantity	NPC Code	PIP Code
	36590019	Transparent Island dressing 5 x 7.2cm	50	ELW545	396-3840
	36590021	Transparent Island dressing 12 x 10cm	50	ELW553	396-3915
	36590022	Transparent Island dressing 8.5 x 15.5cm	50	ELW547	396-3873
	36590023	Transparent Island dressing 20 x 10cm	50	ELW554	396-3923
	36590024	Transparent Island dressing 10 x 25cm	50	ELW546	396-3881
	36590025	Transparent Island dressing 30 x 10cm	50	ELW548	396-3949
	36590026	Transparent Island dressing 8.5 x 9.5cm	50	ELW551	396-3857

