

365 Foam Dressing

Non-Adhesive

PRODUCT CODES

36590061, 36590062, 36590063 & 36590064

Description

365 Foam Dressings are sterile, absorbent foam dressings featuring a hydrocellular pad with a low-friction, breathable polyurethane film backing. 365 Foam Dressings can be used as primary dressings to absorb wound exudate and provide an optimal wound healing environment, even under compression.

Indications For Use

- Chronic and acute wounds with light to moderate levels of exudate
 - Pressure ulcers
 - Diabetic foot ulcers
 - Leg ulcers
 - Post-operative wounds
 - Skin abrasions
 - Superficial and partial thickness burns
 - Donor sites
 - Traumatic wounds
- Shallow, granulating wounds

Precautions

- Foam dressings are not suitable for arterial bleeds.
- Do not overlay or overlap foam dressings.

Instructions For Use

1. Cleanse the wound in accordance with local clinical protocols.
2. Select a dressing large enough to provide a 3cm margin of contact to dry healthy peri-wound skin.
3. 365 Foam Dressings can be cut to fit the wound, however always use a clean technique when cutting the dressing.
4. Prepare and clean the skin surrounding the wound area and remove excess moisture.
5. Remove 365 Foam Dressing from the packaging using a clean technique.
6. Apply the white face to the wound and secure with a secondary dressing, tape or bandage.
7. Always ensure the 365 Foam dressing and any secondary dressings are not applied too tightly. Do not stretch the skin or dressings during application.

Frequency of Dressing Change

- The use of foam dressings should always be re-evaluated if wound exudate levels change.
- During the early stages of wound management, observe the wound daily for signs of infection or other complications (or as recommended by local clinical protocols).
- Infected wounds should be inspected and treated as per local clinical protocol.
- The frequency of dressing change is determined by the condition of the wound and local clinical protocols.
- Dressings can be left in place, undisturbed, for up to 7 days, or until exudate is visible and approaches 2cm from the edge of the dressing, whichever is sooner.

Dressing Removal

Careful and correct dressing removal is vital to minimise the risk of damaging the wound bed and new granulating tissue;

1. Remove the retention dressing or bandage and carefully lift the dressing away from the wound.
2. The dressing should be removed slowly and carefully following the direction of hair growth.
3. As the dressing is removed, the newly exposed skin should be supported.
4. Soiled dressings should be discarded in the appropriate clinical waste receptacle.



Shermond

 Bunzl Retail & Healthcare Supplies Limited. Unit 6 Delta Park Industrial Estate, Millmarsh Lane, Enfield, EN3 7QJ
Tel: +44 (0)1530 278 111
www.shermond.co.uk