



Remove the backing liner marked Tab A




Remove two tape strips from Tab B and fixate to the catheter wings, if required




Whilst holding Tab B and C, apply the dressing (adhesive side down to the site). Ensure the port fits closely around the hub of the catheter. The Transparent film should cover the insertion site.



Remove and retain Tab B which contains the patient documentation label.

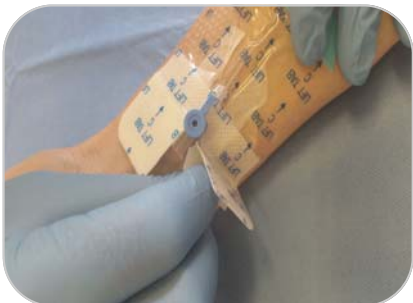

Ensure the dressing is securely in place and remove the tabs marked C on the soft cloth wings. (The soft cloth wings may be overlapped to increase dressing security)

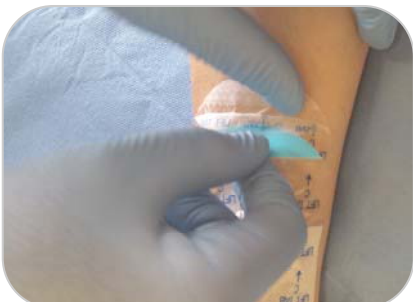

Remove the backing liner marked Tab C, by lifting the blue flap and peeling liner slowly back off the dressing.



Finally, place the documentation label from section B on the skin or the dressing (without obscuring the view of the catheter site).



Gently peel back the non-woven wings to the point at which they reach the film section whilst stabilising the patient's skin and catheter with the other hand.



Take one corner of the film and gently stretch the dressing horizontal to the skin to break the adhesive bond. Repeat this action around the dressing until




Remove the backing liner marked Tab A




Remove centre cut out which also holds the 2 sterile fixation tape strips.



Place the two tape strips over the catheter wings for increased security.

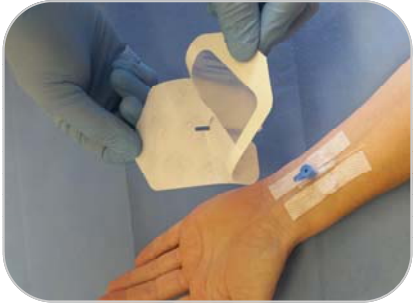

Peel the backing paper away from the framed film dressing.

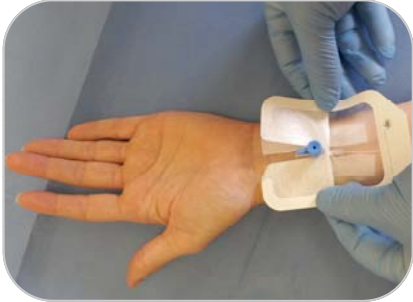

Apply the dressing by placing it over the catheter so that transparent film covers the insertion site



Position the soft cloth tape section over the catheter wings with the port at end of the deep notch. Remove the frame and gently firm edges down to ensure adhesion



Finally remove the documentation label from the frame and apply either on the skin or the dressing



Remove the documentation label. If the film dressing remains stuck down utilise the documentation label by applying it to the corner of the film, it should attach and lift the



Take one corner of the film and gently stretch the dressing horizontal to the skin to break the adhesive bond



Gently peel back the non-woven wings to the point at which they reach the film section whilst stabilising the patient's skin and catheter with the other hand.



Release and repeat the process. Finally remove the two fixation strips