

# ENGLISH

## Instructions For Use - 365 Ported Transparent IV Dressing

### Product Description:

365 IV Dressings are sterile, latex free, self-adhesive, ported IV dressings. 365 IV Dressings consist of a semi-permeable, polyurethane film membrane coated with a patterned hypoallergenic acrylic adhesive, bonded to reinforced soft cloth tips for greater catheter security.

### Dressing features

- Inhibits bacterial growth underneath the dressing due to the breathability of the film.
- Enables visual inspection of the insertion site enabling earlier identification of inflamed blood vessels.
- Allows the patient to shower with the dressing in place.
- Prevention of contamination by dirt or bacteria, thus a reduced risk of wound infection.
- Enhances dressing security and high level of dressing breathability.

### Contraindications and Warnings:

Do not use on patients with a known sensitivity to 365 IV Dressings, or any of its components. 365 IV Dressings should not be used as a dressing for: IV sites only.

- Requires periodical review of the catheter site (to ensure the dressing remains in place particularly post bathing).
- Care should be taken when removing the IV dressing when a replacement is required, especially in the case of fragile or sensitive skin.

### Precautions:

365 IV Dressings should be applied under medical supervision. Always use aseptic techniques. Do not stretch the dressing during application as tension can cause skin trauma. Care should be taken to avoid skin damage with repeated applications, or on patients with fragile skin. When changing the dressing, care should be taken not to dislodge catheters or other devices. 365 IV Dressings are a sterile, single use product. Do not use if the packaging has been damaged.

### Preparation:

Ensure that the skin surrounding the wound is clean, dry, and free from grease, soaps or detergents. Clipping of hair at the site may improve adhesion, shaving is not recommended. Select a dressing that is of a suitable size and shape.

### Dressing Application:

- Clean and dry the application area.
- Remove the sterile dressing from the inner pouch.
- Remove the backing liner marked Tab A.
- Remove two tape strips from Tab B and fixate to the catheter wings, if required.
- Whilst holding tab B and tab C together apply the dressing adhesive side down to the site ensuring the port fits closely around the hub of the catheter and the transparent film covers the insertion site.
- Retain Tab B which also contains the patient documentation label.
- Ensure the dressing is securely in place and remove the tabs marked C on the soft cloth wings.
- The soft cloth wings maybe overlapped slightly to increase dressing security.
- Remove the backing liner marked Tab C, by lifting the blue flap and peeling the liner slowly back off the dressing.
- Finally place the documentation label from section B on the skin or the dressing (without obscuring the view of the catheter site).

### Dressing Wear Time:

The frequency of dressing change will depend on the clinical circumstances. 365 IV Dressings may be left in place for up to seven days. If signs of clinical infection are observed, the dressing should be removed.

### Dressing Removal:

- Gently peel back the non-woven wings to the point at which they reach the film section, whilst stabilising the patient's skin and catheter with the other hand.
- Take one corner of the film and gently stretch the dressing horizontal to the skin to break the adhesive bond.
- Release and repeat the process.

For further information or training please contact your local 365 Healthcare representative.

| Product code | Dressings                        | Size      |
|--------------|----------------------------------|-----------|
| 36590018     | Transparent Adhesive IV Dressing | 7 x 9 cm  |
| 36590020     | Transparent Adhesive IV Dressing | 10 x 12cm |

## Instructions For Use - 365 Transparent Peripheral IV Dressing

### Precautions:

365 IV Dressings should be applied under medical supervision. Always use aseptic techniques. Do not stretch the dressing during application as tension can cause skin trauma. Care should be taken to avoid skin damage with repeated applications, or on patients with fragile skin. When changing the dressing, care should be taken not to dislodge catheters or other devices. 365 IV Dressings are a sterile, single use product. Do not use if the packaging has been damaged.

### Dressing Application:

- Clean and dry the application area.
- Remove the sterile dressing from the inner pouch.
- Remove centre cut out which also holds the 2 sterile fixation tape strips.
- Place the two tape strips over the catheter wings for increased security.
- Peel the backing paper away from the framed film dressing.
- Apply the dressing by placing it over the catheter so that transparent film covers the insertion site.
- Position the soft cloth tape section over the catheter wings with the port at end of the deep notch.
- Overlap each side of the soft cloth tape underneath the port hub.
- Remove the frame and gently firm edges down to ensure adhesion.
- Finally remove the documentation label from the frame and apply either on the skin or the dressing (without obscuring the view of the catheter site).

### Dressing Wear Time:

The frequency of dressing change will depend on the clinical circumstances. 365 IV Dressings may be left in place for up to seven days. If signs of clinical infection are observed the dressing should be removed.

### Dressing Removal:

- Remove the documentation label.
- Take one corner of the film and gently stretch the dressing horizontal to the skin to break the adhesive bond.
- (Tip) If the film dressing remains stuck down utilise the documentation label by applying it to the corner of the film, it should attach and lift the edge of the dressing making dressing removal easier.
- Gently peel back the non-woven wings to the point at which they reach the film section whilst stabilising the patient's skin and catheter with the other hand.
- Release and repeat the process.
- Finally remove the two fixation strips.

For further information or training please contact your local 365 Healthcare representative.

| Product code | Dressings  | Size       |
|--------------|--|------------|
| 36590041     | Reinforced Transparent IV Frame Application Dressing | 5 x 5.7 cm |
| 36590046     | Reinforced Transparent IV Frame Application Dressing | 7 x 8.5 cm |

### Product Description:

365 IV Dressings are sterile, latex free, self-adhesive, ported IV dressings. 365 IV Dressings consist of a semi-permeable, polyurethane film membrane coated with a hypoallergenic acrylic adhesive, bonded to reinforced soft cloth tips for greater catheter security.

### Dressing Features:

- Inhibits bacterial growth underneath the dressing due to the breathability of the film.
- Enables visual inspection of the insertion site enabling earlier identification of inflamed blood vessels.
- The patient to shower with the dressing in place.
- Prevention of contamination by dirt or bacteria, thus a reduced risk of wound infection
- Enhances dressing security and high level of dressing breathability.

### Indications For Use:

365 IV Dressings designed for use as a fixation device to cover and protect vascular access device (VAD) sites.

365 IV Dressings are indicated for peripheral line fixation.

- Prevention of catheter site infection.
- Protection of catheter site from possible contamination or infection.
- Effective management of moisture at catheter site due to superior MVTR dressing properties

### Contraindications and Warnings:

Do not use on patients with a known sensitivity to 365 IV Dressings, or any of it's components.

365 IV Dressings should not be used as a dressing for:

IV sites only.

- Never use a 365 IV dressing on a wound.
- Not suitable for use with patients who are sensitive to acrylic adhesives.
- Not suitable for use on infected intravascular sites.