




## 365 Ported Transparent Dressings

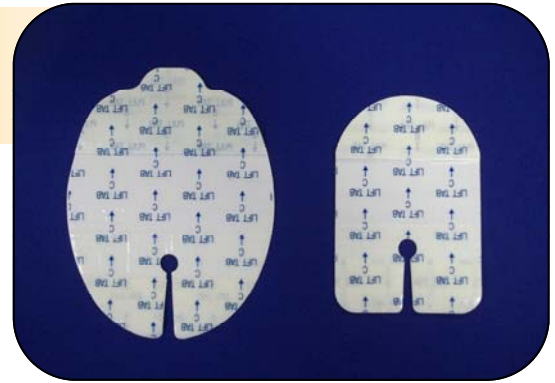

### Product Description

365 IV Dressings are sterile, latex free, self-adhesive, ported IV dressings. 365 IV Dressings consist of a semi-permeable, polyurethane film membrane coated with a patterned hypoallergenic acrylic adhesive, bonded to reinforced soft cloth tips for greater catheter security.

### Key Dressing Features

Transparent film backing

- High Moisture Vapour Transmission Rate of 3000g/M<sup>2</sup>/24hr helps to prevent pooling of fluid under the dressing and premature lifting.
- The breathability of the film inhibits bacterial growth underneath the dressing.
- Allows visual inspection of the insertion site enabling earlier identification of inflamed blood vessels.
- Shower-proof film enables the patient to shower with the dressing in place.
- Impermeable to liquids, bacteria, and viruses.
- Incorporates a patterned adhesive layer specifically designed to provide adequate dressing security combined with high levels of dressing breathability.

Reinforced soft cloth wings

- Provide greater stability around the catheter reducing the risk of phlebitis.
- Supplied with two sterile fixation strips for great catheter and infusion line stability.
- Includes and a sterile documentation label.

### Key Dressing Benefits

- 7 day wear time means fewer unnecessary dressings changes.
- Easy 3 step apply process with A,B,C, labelled release liners and tabs.
- Available for Central Lines and Peripheral IV lines.
- Ensures continuity of care upon patient discharge as 365 IV dressings are also available in the community on Drug Tariff.

365 IV dressings are part of the 365 Healthcare range of Film and Transparent Island dressings

## 365 Transparent Adhesive IV Dressing

### Application Instructions

- Clean and dry the application area.
- Remove the sterile dressing from the inner pouch.
- Remove the backing liner marked Tab A.
- Whilst holding tab B and tab C together apply the dressing adhesive side down to the site ensuring the port fits closely around the hub of the catheter and the transparent film covers the insertion site.
- Remove and retain Tab B which contains the patient documentation label.
- Ensure the dressing is securely in place and remove the tabs marked C on the soft cloth wings.
- The soft cloth wings maybe overlapped slightly to increase dressing security.
- Remove the backing liner marked Tab C.
- Finally place the documentation label from section B on the skin or the dressing (without obscuring the view of the catheter site).

### Removal Instructions

- Gently peel back the non-woven wings to the point at which they reach the film section whilst stabilising the patient's skin and catheter with the other hand.
- Take one corner of the film and gently stretch the dressing horizontal to the skin to break the adhesive bond.
- Release and repeat the process.

### Ordering Information


Product Code	Product Description	Box Qty	NPC Code	PIP Code
36590018	Transparent Adhesive IV Dressing 7 x 9cm	50	ELW541	396-3790
36590020	Transparent Adhesive IV Dressing 10 x 12cm	50	ELW549	396-3816



365 Healthcare Ltd, West Bank, Berry Hill Industrial Estate,  
Droitwich Spa, Unit 1 Worcestershire, WR9 9AX  
[www.365healthcare.com](http://www.365healthcare.com) Tel: 01905 778365 Fax: 01905 826110



## 365 Adult & Paediatric Transparent Adhesive IV Dressings

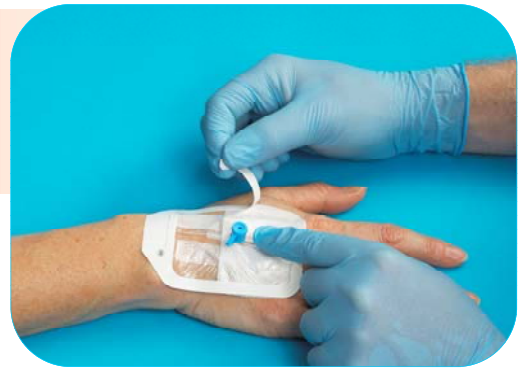

### Product Description

- 365 IV Dressings are sterile, latex free, self-adhesive, ported IV dressings.
- 365 IV Dressings consist of a semi-permeable, polyurethane film membrane coated with a hypoallergenic acrylic adhesive, bonded to reinforced soft cloth tips for greater catheter security.
- Designed for winged and ported catheters

### Key Dressing Features

#### Transparent film backing

- Moisture Vapour Transmission Rate of  $850\text{g}/\text{M}^2/24\text{hr}$  helps to prevent pooling of fluid under the dressing and premature lifting.
- The breathability of the film inhibits bacterial growth underneath the dressing.
- Allows visual inspection of the insertion site enabling earlier identification of inflamed blood vessels.
- Shower-proof film enables the patient to shower with the dressing in place
- Impermeable to liquids, bacteria, and viruses.

#### Reinforced soft cloth wings

- Provide greater stability around the catheter reducing the risk of phlebitis.
- Also supplied with 2 fixation strips for great catheter and infusion line stability

### Key Dressing Benefits

- 7 day wear time means fewer unnecessary dressings changes
- Easy to apply frame delivery application technique
- Includes and a sterile documentation label
- Available for in paediatric and adult sizes
- Ensures continuity of care upon patient discharge as 365 IV dressings are also available in the community on Drug Tariff

365 IV dressings are part of the 365 Healthcare range of Film and Transparent Island dressings

# 365 Adult & Paediatric Transparent Adhesive IV Dressings

## Application Instructions

- Clean and dry the application area.
- Remove the sterile dressing from the inner pouch.
- Remove centre cut out which also holds the 2 sterile fixation tape strips.
- Place the two tape strips over the catheter wings for increased security.
- Peel the backing paper away from the framed film dressing.
- Apply the dressing by placing it over the catheter so that transparent film covers the insertion site.
- Position the soft cloth tape section over the catheter wings with the port at end of the deep notch.
- Overlap each side of the soft cloth tape underneath the port hub.
- Remove the frame and gently firm edges down to ensure adhesion.
- Finally remove the documentation label from the frame and apply either on the skin or the dressing (without obscuring the view of the catheter site).

## Removal Instructions

- Remove the documentation label.
- Take one corner of the film and gently stretch the dressing horizontal to the skin to break the adhesive bond.
- (Tip) If the film dressing remains stuck down utilise the documentation label by applying it to the corner of the film, it should attach and lift the edge of the dressing making dressing removal easier.
- Gently peel back the non-woven wings to the point at which they reach the film section whilst stabilising the patient's skin and catheter with the other hand.
- Release and repeat the process.
- Finally remove the two fixation strips.

## Ordering Information

Product Code	Product Description	Box Qty	NPC Code	PIP Code
36590041	Reinforced Transparent Adhesive IV Dressing 5 x 5.7cm	100	ELW540	396-3774
36590046	Reinforced Transparent Adhesive IV Dressing Frame Application 7 x 8.5cm	50	ELW555	396-3782



365 Healthcare Ltd, Unit 1 West Bank, Berry Hill Industrial Estate,  
Droitwich Spa, Worcestershire, WR9 9AX

www.365healthcare.com Tel: 01905 778365

Fax: 01905 826110