



### Product Description

365 Film is a sterile, latex free, self-adhesive, transparent film wound dressing.

365 Film consists of a semi-permeable polyurethane film membrane coated with a hypoallergenic acrylic adhesive. 365 Film allows visual inspection of the wound site throughout the healing process.

These properties enable:

- Protection of newly forming tissue
- The facilitation of wound healing
- Provision of a 'moist wound healing environment'
- The patient to shower with the dressing in place
- Prevention of contamination by dirt or bacteria, thus a reduced risk of wound infection


365 Film is breathable, allowing good oxygen and moisture vapour exchange with an MVTR of 1000g / 24 hours / m<sup>2</sup>.

It is showerproof and impermeable to liquids, bacteria, and viruses.

### 365 Film Indications For Use:

365 Film can be used as a primary or secondary dressing where there is no exudate, or light levels of exudate.

- Cuts and abrasions
- Clean, closed, post-operative wounds
- Superficial pressure damage
- Leg ulcers
- Protection of donor sites
- Protection against skin breakdown due to friction or continuous moisture exposure
- A fixation device for catheters
- Minor burns

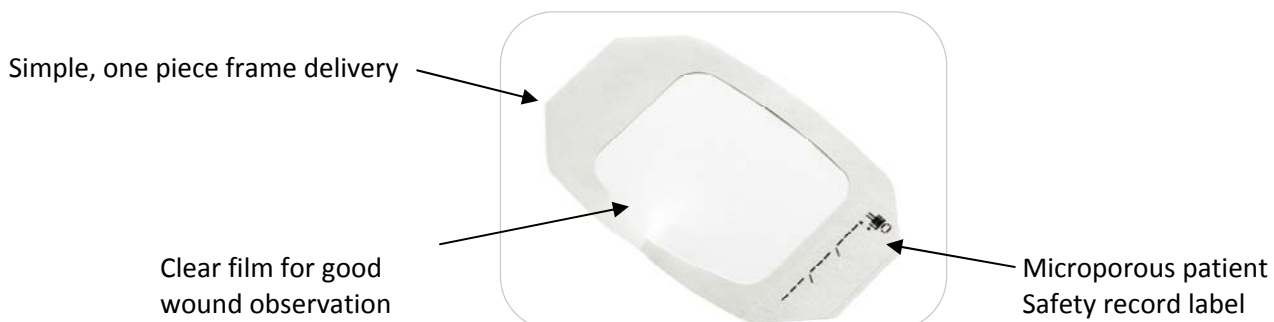

**365 Film is part of a range of dressings that include I.V. Dressings and Transparent Island Dressings**

# 365 Film Dressings

365 Film  
70% cheaper  
than the market leader\*  
\* June 2015 NHS Supply Chain

## Preparation:

Use existing clinical protocols to clean/debride the wound in preparation for the application of 365 Film. Ensure that the skin surrounding the wound is clean, dry, and free from grease, soaps or detergents. Select a dressing that is of a suitable size and shape. Ensure the dressing has a 2cm overlap around the wound edge.

## Dressing Application:

- Peel the liner from the dressing to expose the adhesive surface.
- Position the dressing centrally over the wound bed.
- Press the exposed adhesive side of the dressing onto the skin.
- Slowly remove the frame while smoothing down the dressing edges.
- Then smooth the entire dressing from the centre toward the edges using firm pressure to enhance adhesion.
- When applying to a contoured area, 365 Film may be cut in order to aid adhesion (the bacterial barrier property is compromised if the dressing is cut).



## Dressing Wear Time:

The frequency of dressing change will depend on the clinical circumstances. 365 Film dressings may be left in place for up to seven days. If signs of clinical infection are observed the dressing should be removed.

## Dressing Removal:

- To remove 365 Film from the wound, gently peel and lift one corner of the dressing from the skin.
- Support the skin whilst peeling the dressing off by stretching horizontally (not vertically) and in the direction of hair growth.
- Care should be taken to avoid skin damage with repeated applications, or on patients with fragile skin.
- Use existing clinical protocols to clean the wound in order to remove any remaining exudate residue before wound assessment or the application of further dressings.

## Ordering Information

	365 Film	Product Description	Box quantity	NPC Code	PIP Code
	36590006	Film dressing 4 x 5cm	100	ELW552	396-3725
	36590007	Film dressing 6 x 7cm	100	ELW550	396-3733
	36590008	Film dressing 10 x 12cm	50	ELW542	396-3741
	36590011	Film dressing 10 x 15cm	50	ELW543	396-3758
	36590051	Film dressing 15 x 20cm	25	ELW544	396-3766

