



Wound Closure Strips



Product Description

365 Strips are sterile, hypoallergenic, reinforced, adhesive wound skin closures.

Indications

Superficial cuts and lacerations where wound edges can be approximated.

Precautions

Do not stretch or apply 365 Strips under pressure as blistering may occur as the wound swells.

Strip Features

- Easy to apply and remove
- Adheres to difficult to dress areas and bony prominences
- Associated with better cosmetic outcomes and reduced scarring

Application Instructions

1. Clean wound, as per local protocol, to ensure it is free from grit or foreign bodies.
2. Ensure the application area is clean and dry.
3. Open sterile pouch.
4. Bend and remove the end of perforated backing card to improve ease of application.
5. Remove strip from card using sterile forceps or gloved hand.
6. Place the first strip centrally to align the wound, 90 degrees to the wound direction.
7. Anchor one end of the strip and approximate the wound edges.
8. Fixate the strip across the wound and apply gentle pressure to ensure adhesion.
9. Apply the strips approximately 3mm apart.

For additional security 2 secondary Strips can be placed parallel to the wound to anchor the primary strips in position.


Removal Instructions

1. Where additional anchor strips have been used remove these first.
2. Gently lift the end of the strips.
3. Pull back strips slowly and gently in order to minimise damage to the epidermis.
4. Support the skin with a finger as the strips are removed from the area directly above the laceration.
5. Clean the skin.

Wound Closure Strips



Each dressing is individually

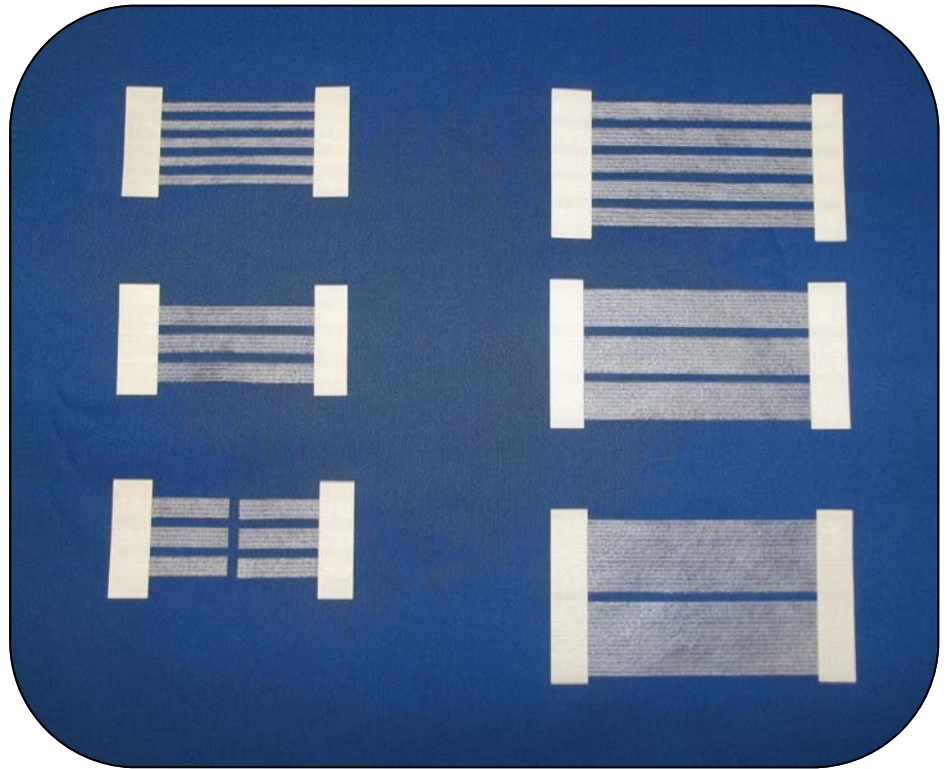

The 3 largest sizes have 2 cards in a pack.

Ordering Information

Product Code:	Product Dimensions:	Box Qty:		NPC Code:
36519064	3mm x 75mm	50	5 strips per card	EIR126
36519065	6mm x 75mm	50	3 Strips per card	EIR124
36519066	6mm x 38mm	50	6 Strips per pouch	EIR125
36519067	6mm x 102mm	50	10 Strips per pouch	EIR123
36519068	13mm x 102mm	50	6 Strips per pouch	EIR122
36519069	25mm x 102mm	50	4 Strips per pouch	EIR127



365 Healthcare Ltd, Unit 1 West Bank, Berry Hill Industrial Estate,
 Droitwich Spa, Worcestershire, WR9 9AX
 www.365healthcare.com Tel: 01905 778365 Fax: 01905 826110