

Remove the dressing backing Paper to expose the adhesive surface.



Ensure the dressing is place centrally over the wound bed  
Take care not to over stretch the wound.



Apply the dressing, slowly removing the frame, whilst smoothing down the dressing edges.



Smooth the dressing from the centre, towards the edges, using firm pressure to enhance adhesion.




Affix patient date label as necessary.



Try to keep the 365 Film in place to allow for maximum efficacy of the dressing.



Hold the edge of the dressing with one hand and thumb. In the opposite hand gently stretch the dressing horizontally, parallel to the skin.

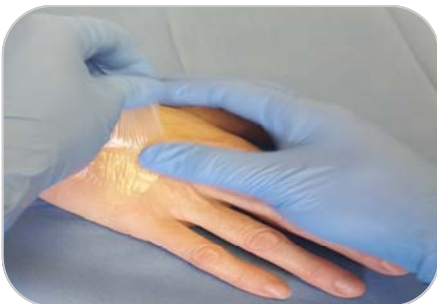

Work around the perimeter of the dressing, until the entire dressing lifts.



Work around the perimeter of the dressing, until the entire dressing lifts. Dispose of the used dressing as per local protocols.